



Injectable Training Model

LF00952U & LF00953U

INSTRUCTION CARD



ABOUT THE SIMULATOR

Nasco developed a lifelike trainer to enhance training for intradermal, subcutaneous, and intramuscular injecting. The material feels like real human flesh. It can be pinched and punctured for injections repeated times.

GENERAL INSTRUCTIONS FOR USE

Dry injecting is recommended. The trainer will reject water or any liquid injected into the trainer. Solvents or corrosive materials will damage the simulator.

SN

Note: Save this card should you need to reference the Serial Number above.

Actual product may vary slightly from photo. Nasco reserves the right to change product color, materials, supplies, or function as needed.

5-Year Warranty

CARE OF THE SIMULATOR


Normal soil can be removed with mild soap and water. Nasco Cleaner will remove stubborn stains. Apply Nasco Cleaner to the soiled area and wipe clean with a soft cloth or paper towel. Allow to dry thoroughly before storing. Never place the simulator on newsprint or any kind of printed paper, plastic, or ballpoint pen. These materials will transfer indelible stains.

AVAILABLE SUPPLIES

| | |
|--------------------|--------------------------------|
| LF01049U | Suture Kit with DVD Collection |
| LF07000U | Suture Replacement |
| LF07000(A)U | Suture Replacement, Pkg. of 5 |
| LF09919U | Nasco Cleaner |
| LF03461U | Optional Carry Case |
| C00879U | Mayo Hegar Needle Holder |

Other Available *Life/form* Simulators

| | | | |
|-----------------|-----------------------------------|-----------------|--------------------------------------------|
| LF00890U | Interactive Suture Trainer, White | LF01042U | Suture Kit |
| LF00892U | Interactive Suture Trainer, Black | LF01046U | Facial Suturing Module Set |
| LF00961U | Intramuscular Injection Simulator | LF01047U | Pediatric Suture Head Kit |
| LF01008U | Intradermal Injection Simulator | LF01131U | Venipuncture & Injection Demonstration Arm |
| LF01028U | Suture & Stapling Practice Arm | LF01184U | Venatech IM & SubQ Simulator, White |
| LF01034U | Suture & Stapling Practice Leg | LF01185U | Venatech IM & SubQ Simulator, Black |
| | | WA18238U | Diabetic Injection Pad |
| | | WA28395U | Injection Belly |

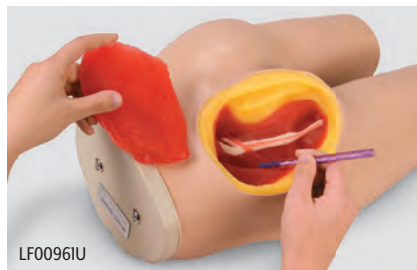

ONLINE: NascoHealthcare.com
E-MAIL: lifeform@enasco.com
PHONE: 1.800.431.4310
ADDRESS: 901 Janesville Ave.
Fort Atkinson, WI 53538